

DigitalScope Technical Requirements

DigitalScopeTM is a web-based whole slide image (WSI) retrieval and viewing system. <u>The current version DSv6.0 does not require the use of Microsoft Silverlight.</u> DigitalScope is supported with Microsoft Internet Explorer 11.0 (limited support for IE 9 and 10) or later, or Fire Fox 4.0 or later, or Safari 3 or the latest Google Chrome version.

DigitalScope Viewer Tool Functions (Tools in italics are displayed if available)			
lcon	Tool		
7	Navigation - Move the image and change the magnification		
(‡)	Full Screen - Toggle on and off the full screen function		
\bigcirc	Case History - View the case history if available		
©	Discussion - View the case discussion if available		
≡	Regions of Interest - View the preselected regions of interest if present		
	Image Adjustments - Adjust the brightness and contrast		
	Overlays - Insert gradient overlays for cell counts		
Φ	Tracker -View thumbnail highlighting the current and previously viewed regions		
3	Measuring Tool - Measure in microns distances between selected points		
P	Link Tool -Generate links to the image with selected attributes		
•	Annotations - Allow administrators to annotate images, not visible with routine viewing		
?	Help - Display additional information		

Navigation Toolbar Functions					
lcon	Tool				
\end{vmatrix}	Home - Return to the 0x magnification				



↑ ← → ↓	Panning - Move the image in the chosen direction The image can also be moved using your mouse to click-and-drag
◆ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦ ♦	Magnification Slider - Adjust the magnification The magnification can also be adjusted by scrolling the mouse wheel
5X 20X 40X 80X 100X	Magnification Buttons - Adjust to a selected magnification

Regions of Interest Toolbar Functions				
Icon	Tool			
(i)	Region of Interest Information - Display information for specific regions of interest			

Image Adjustments Toolbar Functions				
Icon	Tool			
* *	Brightness - Adjust the brightness of the image			
@ @	Contrast - Adjust the contrast of the image			
S	Reset - Reset the brightness and contrast			

Overlays Toolbar Functions				
Icon	Tool			
	Miller Disc - Overlays a Miller graticule for cell counts			
	Grid - Overlay a grid			
- ▼	Color Dropdown - Change the color of the overlay			